




Quick Start Guide

Ventura 3 & 4 Wheel

Please Read Before Operating!

For technical assistance, call toll free (866) 621-4933 or (800) 544-5155.

- 1.** Remove your scooter from its packaging. Loosen the tiller adjustment knob, raise the tiller to the desired angle and retighten the knob.

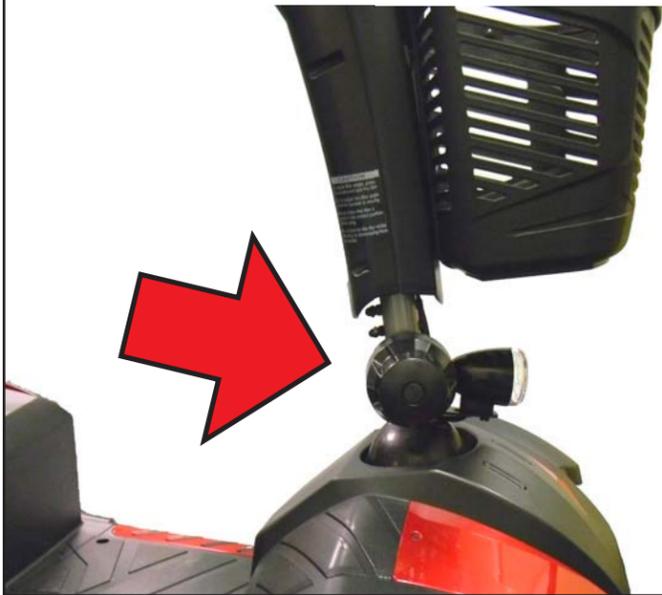


- 2.** By hand, loosen the two knobs that secure the rear shroud to the base of the scooter.



- 3.** Lift up on the shroud to remove.



- 4.** Connect the red connector on the battery harness to the matching red connector on the main cable.

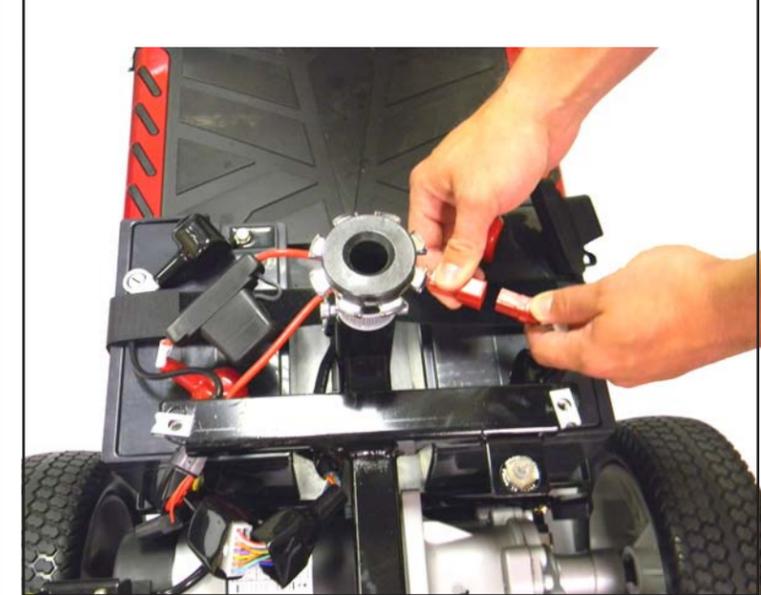

- 5.** If equipped (DLX models), connect the rear light harness on the rear shroud to the main cable on the base of the scooter.



- 6.** Replace the rear shroud and reinstall the two knobs removed in Step 2.



- 7.** While lifting up on the seat release lever, place the seat onto the seat post. Release the lever and rotate the seat until it locks into place.



- 8.** Insert the armrests into the attachment points in the seat frame so that the armrests face forward.



- 9.** Attach the front basket by aligning the bracket on the tiller with the mounting holes on the basket. Then push down to secure in place.



- 10.** If equipped (DLX models), secure the mirrors to the tiller by inserting the mirrors into the connections points on either side of the control panel and rotate clockwise until secured into position.



- 11.** Fully charge the scooter's batteries by inserting the round charger connector into the charging terminal on the scooter's tiller. Then insert the 3-pronged plug into a standard electrical outlet. Be sure to charge the scooter for 8-10 hours before first time use.



- 12.** Before use, make sure that the scooter's yellow freewheel mode lever is in the "drive" position.



Ventura Quick Start Guide



READ BEFORE UNPACKING

This Quick Start Guide offers basic information on initial product setup only. Read the owner's manual and all supplemental information thoroughly before operating the product for information on safety, battery charging, basic operation, comfort adjustments, disassembly and care and maintenance.

NOTE: The illustrations provided may not reflect your actual unit.

For information on the comfort adjustments available on your scooter, please consult your owner's manual.